


ELECTRIC BEAUTY CHAIR

PREMIUM BLACK

MANUAL



Podmiot odpowiedzialny:
MANYBEAUTY Radosław Juszcak
ul. Młyńska 61/2
75-422 Koszalin, Polska
NIP: 669-239-72-28
E-mail: sklep@manybeauty.pl
Strona internetowa: www.manybeauty.pl

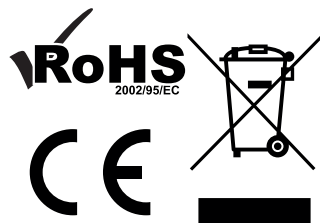

Zgodny z normami : EMC,
LVD, Dyrektywa Maszynowa
2006/42/WE

1. General Information

Manufacturer and responsible entity:

MANYBEAUTY Radosław Juszcak

ul. Młyńska 61/2

75-422 Koszalin, Poland

VAT ID: 669-239-72-28

E-mail: sklep@manybeauty.pl

Website: www.manybeauty.pl

Model: Premium - Black

Intended Use: For professional use in beauty salons, aesthetic medicine clinics, and SPAs.

Maximum Load Capacity: 250 kg

EU Compliance: CE, RoHS, EMC, LVD, Machinery Directive 2006/42/EC

Country of Manufacture: Chin

2. Set Contents

- Chair with electric base (4 motors)
- Removable armrests (2 pcs)
- Extendable headrest with face hole + concealing cushion
- Wired remote control
- Power cable (length: 208 cm)
- 4 leveling feet
- User manual



Parameter	Value
Power	200 W
Net weight	80 kg
Upholstery color	Black
Base color	Black
Packaging dimensions	161 × 63 × 61 cm
Height range	62–91 cm
Total length	200 cm
Seat width	63 cm
Width with armrests	75 cm
Base dimensions	120 × 60 cm
Motors	4 electric motors

4. Main Features and Benefits

- 4 electric motors: adjustment of height, backrest angle, seat angle, and leg rest elevation
- Height adjustment: 62–91 cm
- Length in lying position: 200 cm
- Full transformation into a bed
- Ergonomic remote control – easy and intuitive to use
- Adjustable headrest (6 levels): extendable up to 22 cm
- Upholstery made of black synthetic leather – easy to clean
- Decorative armrests with wooden elements
- Base with leveling feet – stability on various surfaces

5. Assembly and First Use

- Carefully unpack the chair, paying attention to delicate parts and finish.
- Place the chair on an even surface.
- Screw in the 4 leveling feet.
- Connect the power cable to a 230 V socket.
- Mount the armrests and headrest.
- Make sure all components are stable.
- Check the functionality of the controls.
- Test each function separately before first use with a client.

6. Operating Instructions

- Before each use, check the chair's stability.
- Ensure the client is seated or lying evenly – avoid one-sided overloading.
- Do not exceed the maximum load of 250 kg.
- Do not sit or stand on moving parts (e.g., the leg rest).
- Use only indoors, in dry environments.
- Wipe surfaces with a damp cloth and a PVC-safe disinfectant.

7. Maintenance and Cleaning

- Clean upholstery with a soft cloth and a mild detergent suitable for PVC surfaces.
- Do not use corrosive agents or alcohol.
- Regularly check the functionality of the remote and cables.
- If any structural looseness is observed – immediately contact the responsible entity.

8. Warnings and Safety Precautions

- Electrical device – do not use near water!
- Do not open the motor or control panel casings!
- All warranty repairs must be carried out by the responsible entity or an authorized service provider.
- Keep the device away from children.
- Disconnect the chair from power after use.
- Before transport or moving, disconnect from power and secure all movable parts.

9. Disposal

This product is marked with the symbol for selective collection of electrical and electronic waste. Do not dispose of it with regular household waste. Deliver it to an appropriate electronic waste collection point in accordance with the WEEE Directive (2012/19/EU).

10. Warranty and Service

The product is covered by a 12-month manufacturer's warranty.

All complaints should be sent to: sklep@manybeauty.pl with a description of the issue and proof of purchase.

For many other
products visit



www.manybeauty.pl